



Detach & Go Grooming & Healthcare Kit 49612

Detach & Go Travel Case

To fit cases together, separate cases, position with side flaps opposite each other. Press side flap of case A to side of case B (Figure 1). With side flap attached, position case A on top of case B. Press side flap of case B to side of case A (Figure 2).


To clean: Wash with a damp cloth.

Digital Thermometer with Protective Case

The Safety 1st Digital Thermometer takes a clear and accurate temperature without the worries of a glass/mercury device. Effective for oral, rectal or underarm use.



Hospital's Choice

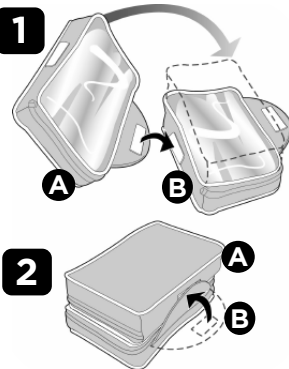

How To Care For This Product:

- Use of probe covers is recommended to help avoid spreading germs and to keep the thermometer clean. The thermometer should be cleaned after each use even when probe covers are used.
- Clean the thermometer tip and probe using isopropyl alcohol or soap and water.
- Store the unit in the protective case while not in use.
- Do not store or keep in direct sunlight, or in high temperature areas.
- Do not drop the thermometer, as this may ruin the unit.

What Is A "Normal" Temperature?:

Temperature readings vary in different individuals. A normal temperature refers to each individual's average temperature while they are well. Normal average temperatures range from 97°F (36.1°C) to 100°F (37.7°C) when taken orally. From an oral temperature, you may calculate a rectal equivalent temperature by adding 1°F (0.5°C) while an underarm temperature requires subtracting 1°F (0.5°C). For example, if your oral temperature is 98.6°F (37°C) this is the same as a rectal temperature of 99.6°F (37.5°C) or an underarm temperature of 97.6°F (36.4°C).

How To Take Temperature:

- 1 Select rectal, oral or underarm as your measurement site based on the individual requiring a temperature measurement.
- 2 Push the on-off button located on the cap. The thermometer display will read: "188.8 °F". This is a function check indicating the unit is working properly. The thermometer will read L°F (or L°C) and begin measuring temperature when exposed to body heat.
- 3 The °F (or °C) symbol will continuously flash until completion of the temperature taking process. Please note that the "▼" indicates a low battery. An "ERR" message in the window indicates your unit is not functioning properly. Do not expect an accurate temperature reading if this low battery indicator is present.
- 4 When peak temperature is reached, the °F (or °C) symbol will stop flashing.
- 5 Read and record the temperature and time for reference.
- 6 Push the on/off button to turn the unit off. If you do not turn the unit off, it will shut off automatically in approximately 10 minutes.

Rectal Use:

- NOTE:** To purchase thermometer covers call Consumer Relations at (800) 544-1108, and ask for part number 49520.
- Use of thermometer cover is recommended for rectal use.
 - Clean the thermometer as recommended.
 - Commonly used for babies, young children or when it is difficult to take an oral or underarm temperature.
 - Lubricate the thermometer cover with a water-soluble lubricant or petroleum jelly.
 - Gently insert the probe tip no more than 1/2 inch or 1.3 cm into the rectum. Do not force the tip into the rectum if resistance is encountered.
 - The peak temperature should be reached in approximately 60 seconds.

Oral Use: (For children 4 years and older)

- Do not drink hot or cold fluids, exercise or perform other activities that will raise or lower temperature readings when compared to your normal, average temperature. The mouth should remain closed up to 5 minutes before attempting a reading.
- Place the probe tip well under the tongue. (See diagram) as indicated by the "G". Instruct children not to bite down on the probe.
- The peak temperature should be reached in approximately 60 seconds. Opening the mouth could result in a longer time for a reading.



Underarm (Axillary) Use:

- Wipe underarm with a dry towel.
- Place probe tip in armpit and bring arm down against the body. The thermometer should be pointing in the same direction as your body if possible.
- The time required for an accurate underarm temperature may be 90 seconds or longer depending on placement of the probe and closure of the arm against the body.




Battery Information:

When the "▼" appears in the lower right of the display, the battery is exhausted. Dispose of the thermometer as the battery is non-replaceable. Be aware of any local regulations concerning disposal of this product. The battery is well sealed inside the thermometer to reduce the risk of being swallowed by children.

Measurement Range: 89.6°F to 111.2°F / 32°C to 44°C.

Measurement Accuracy: 0.2°F / 0.1°C between 93.2°F to 107.6°F / 34°C to 42°C at an ambient temperature of 64.4°F to 82.4°F / 18°C to 28°C.

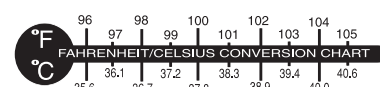

One Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Digital Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on front page and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or inconsequential damages.

Healthcare & Grooming Essentials

Using Medicine Spoon & Syringe

- **Child should sit upright when taking medicine. Hold baby upright when administering medicine.**
- **Administer medicine slowly enough so your child may swallow comfortably. Pause and repeat if necessary.**
- **Wash thoroughly before first use and after each use.**

CAUTION: KEEP ALL MEDICATIONS OUT OF CHILD'S REACH. WE RECOMMEND THAT YOU USE A CHILD RESISTANT TOP TO SEAL THE MEDICINE BOTTLE AND STORE IT IN A LOCKED CABINET OR OTHER LOCATION THAT IS NOT ACCESSIBLE TO A CHILD.

Easy Fill Medicine Spoon

The **Easy Fill Medicine Spoon** stands securely on its own as you fill it with medicine. The soft rounded edge is especially gentle on your child's mouth and the attached plug is helpful in preventing spills. Lay it down horizontally between uses and the stay-clean legs keep the spoon from touching the table.

- 1 Stand medicine spoon on counter and pour medicine to prescribed dosage.
- 2 Plug medicine spoon with stopper to prevent spillage.

To clean: Wash in warm soapy water and rinse thoroughly. Top-rack dishwasher-safe.

Easy Fill Medicine Syringe

Our **Easy Fill Medicine Syringe** is a convenient way to administer medicine to infants and toddlers. The curved, Soft Touch grip allows for comfortable finger positioning and easy dispensing.

- 1 Insert syringe tip into medicine.
- 2 Draw back plunger slowly to fill tube with medicine. Stop when medicine reaches prescribed dosage.
- 3 Insert the tip of the dispenser into your child's mouth. Point tip toward child's cheek.
- 4 Gently push plunger in to release liquid. Be sure to push slowly enough so your child may comfortably swallow. Pause and repeat if necessary.

To clean: Wash all parts separately. Wash with warm, soapy water and rinse thoroughly. Repeat after administering each dosage.

Clear Tip Nasal Aspirator

Our **Clear Tip Nasal Aspirator** quickly and conveniently removes excess mucus so your baby can breathe comfortably again. Parents will enjoy the built-in notches for proper finger placement and the clear removable tip that allows you to see when the aspirator is working.

- 1 Place middle and index fingers on finger grooves and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To clean: Remove aspirator tip from bulb for easy cleaning. Wash in warm soapy water and rinse thoroughly with warm, clean water. Top-rack dishwasher-safe. Avoid using abrasive pads or cleaners. Repeat after administering each dosage. DO NOT place in microwave.

CAUTION: NEVER push the aspirator too far into the nostril and always dry thoroughly after each use. NEVER use in baby's mouth or ears. When storing, keep away from heat and bright light.

Gentle Care Brush & Comb Set

The **Gentle Care Brush & Comb Set** features large, Soft Touch handles so parents and kids get a secure and comfortable grip. The hairbrush's soft bristles are extra gentle on baby's sensitive scalp while the comb's dual-teeth design lets you adjust for the thickness and dampness of your child's hair.

To clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

No Scratch Mittens

The **No Scratch Mittens** are designed to help prevent infants from accidentally scratching themselves. The soft cotton material is comfortable for baby and the gentle elastic wristband helps keep them securely in place. **Use and Care:** Machine washable. Line dry.

Baby Scissors

Keeping your child's nails neat and clean can sometimes be difficult. Our **Baby Scissors** help make it easy. They are specially designed with curved cutting edges to easily trim little nails. The easy-grip handle ensures a secure grip during use.

To clean: May be sterilized by wiping edges with rubbing alcohol. Rinse in warm, clear water and wipe with a soft, dry cloth.

CAUTION: Be careful not to trim too close to baby's sensitive skin.

Steady Grip Nail Clippers & 12 Emery Boards

Moms and dads will love the extra-large handle of the **Steady Grip Nail Clippers**. Its unique Soft Touch design ensures a comfortable grip while the curved cutting edges effectively trim baby's nails. Comfortably smooth and file baby's little nails with our emery boards.

- 1 Lift leverage arm from handgrip and rotate 180°.
- 2 Position nail between cutting edges and squeeze handle to cut nail.
- 3 Smooth nails as often as necessary using emery board.

To clean: Wipe with a damp cloth and dry after each use. Clippers may be sterilized by wiping with rubbing alcohol.

NOTE: It is best to trim after a bath when nails are soft, or at night when baby is sleeping.

Tweezers

Use and Care: Helpful for grasping small items. Use as necessary. Clean with rubbing alcohol.

Toddler Toothbrush

The contoured, Soft Touch handle of the **Toddler Toothbrush** makes it easy for small hands to grip. Its soft, fine bristles are especially gentle on your toddler's sensitive teeth and gums. Place it on the table or sink and the convenient, roll-resistant handle keeps it from falling off.

Your child may begin to use the **Toddler Toothbrush** once all primary teeth are in or when recommended by your dentist. With a pea-sized amount of toothpaste, let your child brush for 2-3 minutes using an up and down motion. Continue to help and supervise your child during brushing – do not allow child to swallow toothpaste. **NOTE:** Toothbrushes should be replaced every 2-3 months.

To clean: Rinse each brush thoroughly before first use and after each use. **Important Note:** Questions about your child's oral health should be answered by your family dentist.

CAUTION: Adult supervision required. Do not allow child to run or walk while carrying or using toothbrush. Do not let children play with toothbrushes.

CAUTION:

THESE PRODUCTS ARE NOT TOYS. Do not allow children to play with them. To prevent accidental injury, your child should never be left unattended. Products should only be used with adult supervision. When not in use, keep out of reach of children.

©2008 Dorel Juvenile Group. All Rights Reserved.

www.djgusa.com (800) 544-1108

www.safety1st.com

Made in CHINA. Fabriqué en CHINE.

Styles and colors may vary. Les styles et les couleurs peuvent varier.

Distributed by (Distribué par) Dorel Juvenile Group, Inc.,

2525 State St., Columbus, IN 47201-7494

Dorel Distribution Canada, 873 Hodge, St. Laurent, QC H4N 2B1

12/25/08 4358-4533

